SD BIO LINE HIV-1/2 3.0

(03FK10, 03FK16)

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1

EN Now, open the package and look for the following:

1. Test devices individually foil pouched with a desiccant
2. Assay diluent
3. Instructions for use

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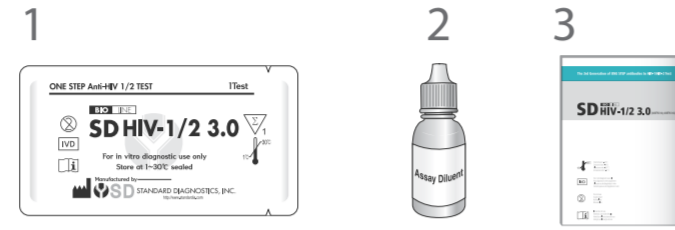
EN Open the foil pouch and look for the following:

1. Result window
2. Sample well & Assay diluent well

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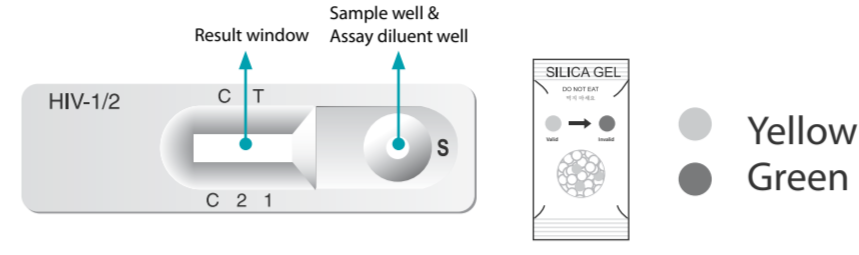
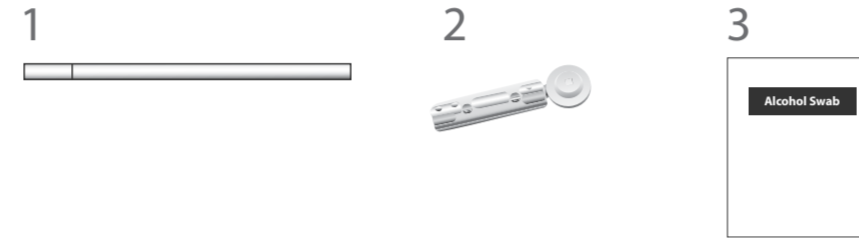
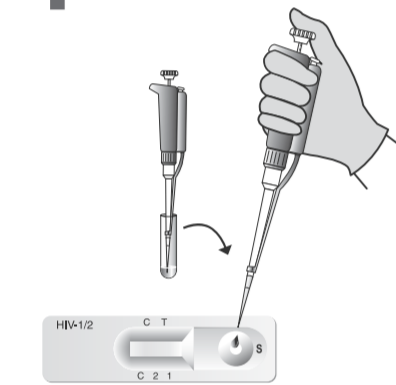
4

10µl (Plasma or Serum) Or 20µl (Blood)

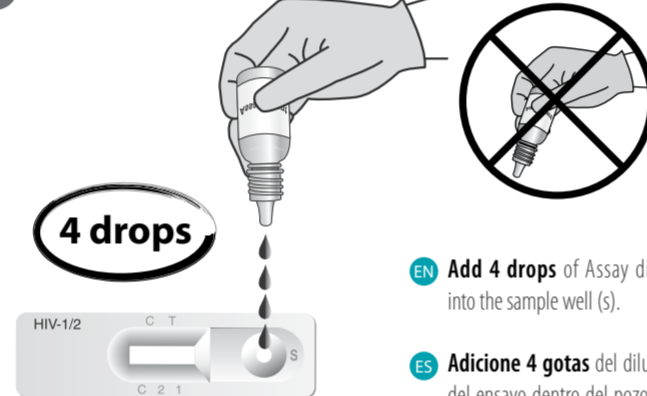
EN Add 10µl of plasma or serum (20µl of blood) into the sample well (S) using a micropipette.

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5

4 drops

EN Add 4 drops of Assay diluent into the sample well (S).

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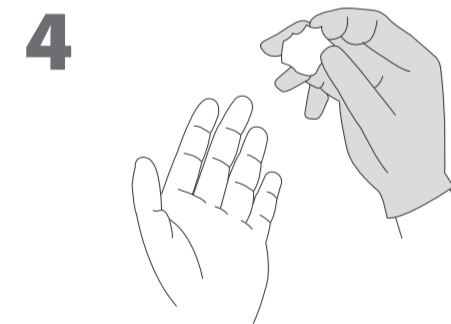
4

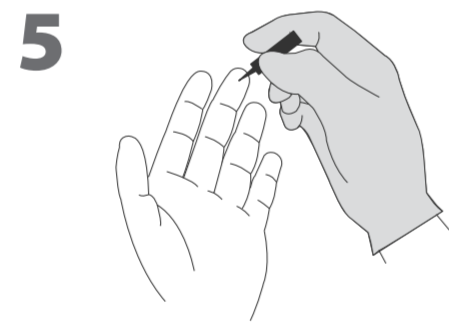
EN Clean the area to be lanced with an alcohol swab.

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PT Limpe a área a ser lancetada com swab de álcool.

5

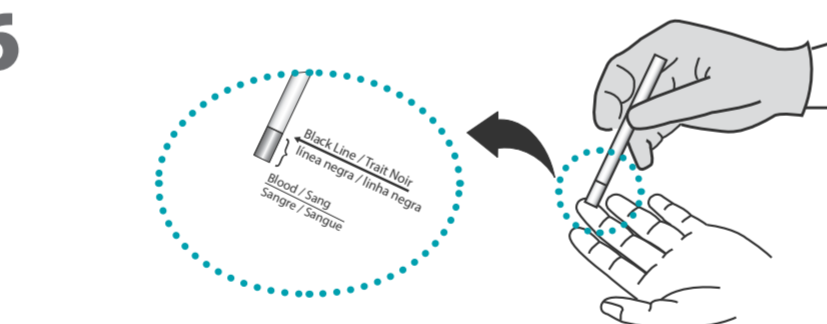
EN Pierce with a sterile lancet provided.

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PT Fure com a lanceta estéril contida no kit.

6

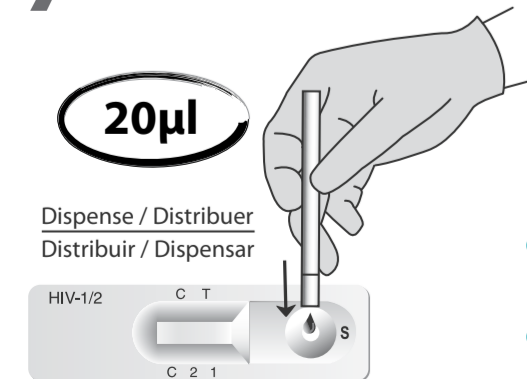
EN Take a 20µl capillary pipette provided, immerse the open end in the blood drop and then release the pressure to draw blood into the capillary pipette to **black line**.

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ES Distribuir / Dispensar

PT Distribuir / Dispensar

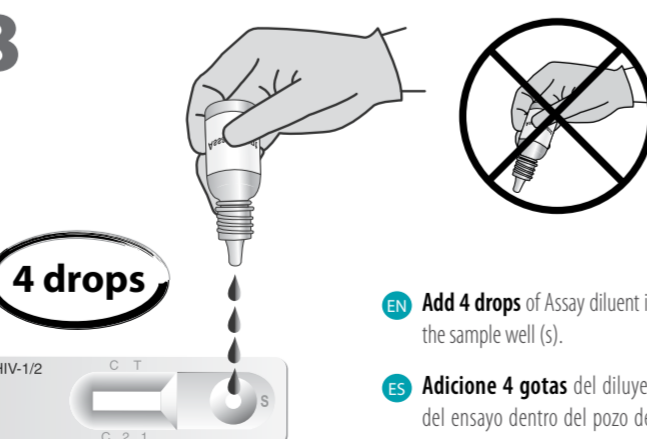
EN Add 20µl of drawn blood into the sample well (S).

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PT Adicione 4 gotas de diluente de ensaio na janela de amostra.

2

EN FIRST, read carefully the instruction on how to use the SD BIOLINE HIV-1/2 3.0 test kit.

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3

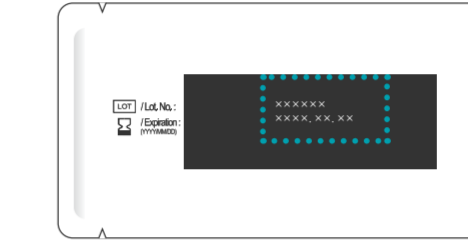
EN Next, look at the expiry date at the back of the foil pouch. Use another kit, if expiry date has passed.

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[For example / Par exemple / Por ejemplo / Par exemplo]

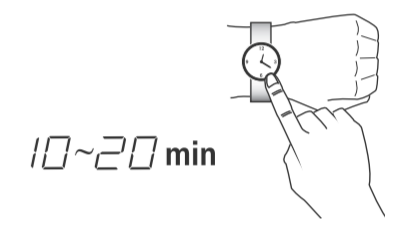
Interpretation / Interprétation / Interpretación / Interpretação

EN Time to result is 10 to 20 minutes. After adding the diluents, read the result after 10 minutes but not more than 20 minutes. If the test result is not legible after 10 minutes due to high background color, read again later but within 20 minutes of adding the diluents. Do not read after 20 minutes.

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FR Weak / Floue / Débil / Fraco

3 LINES / 3 BANDES / 3 LÍNEAS / 3 LINHAS

EN When 3 lines appear "C", "1" and faint "2" line

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EN Bande "1" is darker than line "2", it should be interpreted as HIV-1 positive only. (Not HIV-2 positive).

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EN Bande "2" is darker than line "1", it should be interpreted as HIV-2 positive only. (Not HIV-1 positive).

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EN Bande "1" and "2" are both faint, it should be interpreted as HIV-1 and HIV-2 positive. (Both HIV-1 and HIV-2 positive)

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EN Bande "1" and "2" are both dark, it should be interpreted as HIV-1 and HIV-2 positive. (Both HIV-1 and HIV-2 positive)

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EN Bande "1" and "2" are both very faint, it should be interpreted as HIV-1 and HIV-2 positive. (Both HIV-1 and HIV-2 positive)

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EN Bande "1" and "2" are both very dark, it should be interpreted as HIV-1 and HIV-2 positive. (Both HIV-1 and HIV-2 positive)

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EN Bande "1" and "2" are both very faint and very dark, it should be interpreted as HIV-1 and HIV-2 positive. (Both HIV-1 and HIV-2 positive)

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